

For use in

- Laparotomy
- Laparoscopy
- Hysteroscopy



Instructions for use

1. Bring the product to room temperature, open the pouch and transfer the syringe onto the sterile area, adopting normal aseptic techniques used in surgical theatre.
2. Remove the protective cap on the tip of the syringe and attach the enclosed cannula.
3. Push the plunger and apply the gel inside the abdomino-pelvic cavity.
4. Cover the areas of interest completely with the gel. Even if the thickness of the gel applied does not influence efficacy of the product, it is recommended to apply a 1-2 mm thick layer of gel.
5. Do not irrigate the surgical area after the gel is applied.

Storage

Keep refrigerated (2-8° C). Do not freeze. Shelf life is three years.

Product that has been warmed up to room temperature without being used can safely be refrigerated again up to 10 times if the primary package is not broken.



About Hyalobarrier

Hyalobarrier is indicated to prevent or reduce the post-surgical adhesion formation in the abdomino-pelvic area. Hyalobarrier is a biabsorbable, transparent and highly viscous gel, composed of hyaluronic acid autocross linked without the introduction of any foreign substances.

Thanks to its viscosity, Hyalobarrier adheres perfectly to the tissue surface and to the abdominal wall, creating an anti-adhesion barrier which keeps the adjacent tissues separated during the repair phase subsequent surgical procedure.¹⁻⁴ The gel is completely degraded following the natural pathway for hyaluronan, within seven days after the application.⁵

Contraindications

Known hypersensitivity to the product, like other implant materials, the device must not be used in patients with active infection or contamination of surgical site.

Ordering information

Hyalobarrier can be ordered directly from your pharmacy. Use below product number. Due to cold transport of the product minimum ordering quantities are the following:
Norway: 10, Denmark: 10, Finland: 10, Sweden: 13

Hyalobarrier gel endo (816989)

Indicated for use in laparoscopy and hysteroscopy surgery procedures. Package includes: One individually packaged, single use sterile syringe containing 10 ml of gel. One individually packaged single use, sterile 30 cm cannula. Suitable for use in a 0,5 cm trocar.

References

1. Revaux A, et al. *Gynecol Obstet Fertil.* 2008 Mar;36(3):311-7.
2. Acunzo G, et al. *Hum Reprod.* 2003 Sep;18(9):1918-21
3. De Iaco PA, et al. *Fertil Steril.* 1998 Feb;69(2):318-23.
4. Mensitieri M, et al. *J Mater Sci: Mater Med* 7, 695–698 (1996)
5. HYALOBARRIER® Instructions for use. HYALOBARRIER® Technical dossier. Anika Therapeutics

Contact information

Fredrik Ahlgren, Nordic Marketing Director
Phone +46 40-36 66 22, Mobile +46 76-25 16 86
E-mail: fredrik.ahlgren@nordicpharma.com